



Directions For Use



- Store Hy-Tape in a climate controlled area that neither exceeds nor goes below average room temperature.
- Clean dry surface prior to application. Alcohol wipe may be necessary in cases of oily skin.
- No skin prep needed to assist adhesive performance.
- Peel the appropriate width and length of Hy-Tape product for its specific application.
- Apply firmly to skin and appliance. Smooth all edges of Hy-Tape to the surfaces it is attached to.
- To remove, start at an edge, lift slightly and peel back slow and low across the area it covered, in the same direction the hair is growing. Low angle peels will reduce tension on the skin causing less trauma upon removal.